




The complete solution for the hearing health of professionals exposed to noise

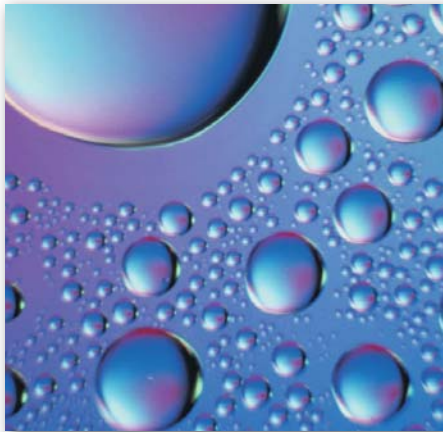

Hygiene and maintenance of the protections

A bulb for checking and maintaining the filter

A Cleanil solution, effervescent, for deep down cleaning

Wipes for superficial maintenance of the protections

Daily wear of the hearing protections results in soiling: dust, wax deposits... These substances accumulate on the protection, but particularly in the canal and filter. Regularly maintaining your hearing protections means ensuring comfort in use, but also maintaining the efficacy of the acoustic filter by avoiding excessive insulation and allowing communication.

For 5 you key points

- Maintenance and monitoring of the efficacy of your acoustic filters for 100% safety using the CLEANIL Plus bulb.
- Ease of use of the CLEANIL effervescent tablets for weekly care.
- Simplicity of the 1st insertions of the protections in the ear using the COMFORTIS device.
- The Excellium and Optimum packs: individualised packaging for one year's maintenance of your protections.
- Maintenance of the protections using the 100% biodegradable wipes to ensure that your protections are pleasant to wear all day long.

Description



A complete and easy to use range of maintenance products is available to ensure the **efficacy** of your COTRAL hearing protections. Supplied in an **individual one-year pack**, your daily and weekly maintenance can be performed with **complete peace of mind**:

- The **Optimum and Excellium "first equipment" packs** match the degree of use of your protections and your environment.
- The **"refill" packs** allow you to benefit, year after year, from the Cleanil tablets and wipes necessary for maintenance of your protections.

	Cleanil Pro pack	48 Cleanil tablets	Sachet containing 40 wipes
PRIMO equipment PACK Excellium	1	1	6
PRIMO equipment PACK Optimum	1	1	3
REFILL PACK Excellium	X	1	6
REFILL PACK Optimum	X	1	3

Your guarantees

- By using the Cleanil solution, you will **maintain the efficacy of the acoustic filter** throughout the entire duration of the guarantee.
- A after-sales service at www.cotral.co.uk "customer area" or by telephone at **+ 33 (0) 2 31 69 05 13**.





Hygiene and maintenance of the protections

To ensure that your protections remain **comfortable**

To maintain the **efficacy of the acoustic filter**

To encourage wearing of the protections **100% of the time spent exposed to noise**

Hygiene of the protections according to standard EN 458

This European standard guides companies in improving wearing of the hearing protections.

Chapter 7 refers to precautions for use and maintenance of the individual protections and recommends regular maintenance and cleaning of one's protections in order to avoid any loss of attenuation.

Contamination of the protections with foreign substances, solutions, liquid residues, dust and particles, etc. may result in skin irritation or abrasion. The user is recommended to ensure that his/her hands are clean when handling the protections.

The earpieces should be cleaned after use according to the manufacturer's instructions. Reusable earplugs should be cleaned carefully according to the manufacturer's instructions and subsequently stored in a clean case until their next use.

1 For the 1st insertions in the ear

Apply a drop from the Comfortis device to the tip of the protection and spread slightly. This cream will make insertion of the protection into the auditory canal more pleasant and will allow correct positioning in the ear.



2 For maintenance of the acoustic canal of the protections

Once a week, immerse the hearing protections in the Cleanil solution (1 tablet to be dissolved in water in the beaker provided for this purpose) and allow to act for 2 hours.



Dry the acoustic canal using the bulb and ensure that air circulates in the filter as a sign of proper operation.

3 For regular maintenance of the protections

Clean the outer surface of your protections at the end of each working day to ensure that they remain clean and pleasant to wear.



Laboratoire COTRAL
BP100 14110 CONDE SUR NOIREAU
☎ +33 (0) 2 31 69 36 36
☎ +33 (0) 2 31 69 36 35
www.cotral.com

Labor COTRAL
Lebacher Straße 4 D-66113 SAARBRÜCKEN
☎ +49 (0)6 81 9963 637
☎ +49 (0)6 81 9963 621
www.cotral.de

Laboratorium COTRAL
8 Avenue des Arts B-1210 BRUXELLES
☎ + 32 (0)2 545 11 52
☎ +32 (0)2 545 11 46
www.cotral.be

Labor COTRAL
Bahnhofstrasse 52 CH-8001 ZÜRICH
☎ +41 (0)44 214 67 24
☎ +41 (0)44 214 65 19
www.cotral.ch



Laboratoire COTRAL
ZA Charles Tellier
F.14110 CONDE SUR NOIREAU